

Procedure

1. Serum or Plasma: Hold the dropper vertically and transfer 1 drop of serum or plasma to the specimen well (S) of the Test Cassette and add 1 drop of buffer, then start the timer.

2. Venipuncture Whole Blood: Hold the dropper vertically and transfer 2 drops of whole blood to the specimen well (S) of the Test Cassette and add 1 drop of buffer, then start the timer.

3. Fingerstick Whole Blood: To use a capillary tube: Fill the capillary tube and transfer approximately 50 mL of fingerstick whole blood specimen to the specimen well (S) of the Test Cassette, then add 1 drop of buffer and start the timer.

4. Wait for the coloured line(s) to appear.
Read results at 5 minutes. Do not interpret the result after 10 minutes.

